



SCIENCE. LOGISTICS. CERTAINTY.

EXP-6 IVF Shipment: Unloading**Sample Receiver****How to Unload Egg, Sperm and Embryos****Side 1****Step 1**

Wear Safety Gloves.
While holding on to the black strap
Remove the shipper plug
Pull shipper plug upwards and out.

Step 2

Place the shipper plug to the side.


When the Commodity is ready to be relocated to another cryogenic storage tank; pull up on the black strap.

Step 3A

If the straw/commodity is loaded in a **65 mm** goblet (Daisy Goblet), pull up on the black strap to expose the top of the goblet; remove goblet.

**Step 3B**

If the straw/commodity is loaded in a **13 mm** goblet on a Cane, pull up on the black strap to expose the top of the Canister and remove the Cane.

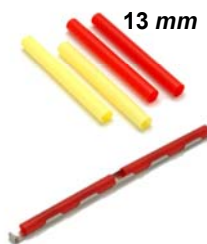


Replace the Lifter or Canister back into the dewar to be returned to CryoPort.

Step 4

Replace shipper plug.
Push plug fully into place.

Step 5

If any label was used in this area for your shipment please place the enclosed "EMPTY" label over it.

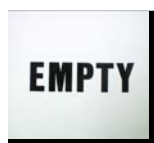**Step 6**

Remove only the Stop Insert from inside the shipping pouch on flap "B". Throw away ONLY the Stop Insert.

Make sure required documents are left in the shipping pouch.



**Instructions Continued
on Back of Sheet.**

**24-HOUR HELP LINE
(949) 470-2305**



SCIENCE. LOGISTICS. CERTAINTY.

EXP-6 IVF Shipment: Unloading**Sample Receiver****How to Unload Egg, Sperm and Embryos****Side 2****Step 7**

Close flaps making sure to have flap "B" showing on the outside of the white carton.

FOR INTERNATIONAL SHIPMENTS ONLY:

- REMOVE AND SIGN ALL FOUR (4) COPIES OF COMMERCIAL INVOICES.
- Place all four (4) signed copies of Commercial Invoices back inside shipping envelope on flap "B" behind Air Waybill.

Step 8

Seal top of white carton with packaging tape making sure to have flap "B" showing on the outside of the white carton.

**Please return
empty shipper
to CryoPort
within 24 hours.**



**24-HOUR HELP LINE
(949) 470-2305**

Cryoport, Inc. • 17305 Daimler • Irvine, CA 92614

Phone: 1-949-470-2300 Option 1- IVF Customer Service

Fax: 1-949-544-8814 • www.cryoport.com • E-mail: ivf-cs@cryoport.com